

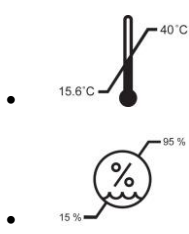
## Product Data Sheet Feverscan® Disposable Products

**Product Description: Disposable Forehead Temperature Trend Monitors**

**Product Codes: P430, P431, P434, P436, P492**

Code #	Temperature Range	Core Adjusted Offset
P430	94-104°F	4.0°F
P431	35-40°C	2.2°C
P434	84-106°F	Varied
P436	94-104°F	4.0°F
P492	94-105°F/34.5-40.5°C	4.0°F / 2.2°C

### Construction:

Substrate	Polyester
Adhesive	Hypoallergenic, pressure sensitive acrylate Carrier: Transparent 3 mil polyethylene film Release Liner: Bleached Kraft-Glassine paper, silicone coated on both sides, 3.5 mils Operating temp: -1 to 82°C (continuous use)
Latex	Not made with natural rubber latex
Color Change Material	Non toxic mixture of liquid crystal esters
Accuracy	± 1.0°F / 0.6°C
Storage & Handling	 <ul style="list-style-type: none"> <li>• Store at 60 to 104°F (15.6 to 40°C)</li> <li>• Store at 15 to 95% RH</li> <li>• Optimal storage at 60 to 80°F (15.6°C to 26.7°C) in 40 to 60% RH</li> </ul>

### How to Read

1. Remove backing liner and apply monitor to center of forehead, just above the eyebrows. Skin surface should be clean and dry.
2. Read monitor when the boxes stop changing color (generally 15 seconds), and while it is still on the forehead.
3. Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.

### Intended Use

- Measure human forehead surface temperature and monitor temperature trends

# Feverscan Reusable Product Data Sheet

**Product Description:**

Reusable Forehead Temperature Monitor

Product Codes: P432, P433

<b>Code #</b>	<b>Temperature Range</b>	<b>Core Adjusted Offset</b>
<b>P432</b>	94-104°F	4.0°F
<b>P433</b>	35-40°C	2.2°C

**Construction:**

Substrate	Polyester
Color Change Material	Non-toxic mixture of liquid crystal esters
Latex	Not made with natural rubber latex
Accuracy	± 1.0°F / 0.6°C
Storage & Handling	Store at 60 to 104°F in 15 to 95% RH Optimal storage at 60 to 80°F in 40 to 60% RH

**How to Read**

1. Apply to center of forehead just above the eyebrows and hold in place
2. Read monitor when the boxes stop changing color (generally 15 seconds), and while it is still on the forehead.
3. Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.

**Intended Use**

- Measure human forehead surface temperature and monitor temperature trends