


BriteLine®

Urine Specimen Temperature Verification Thermometer

A comprehensive system of thermometers based on proprietary formulations of liquid crystals that change color at a designated temperature.

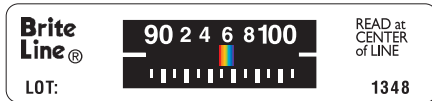

BriteLine® Label

Description	Disposable with Adhesive Label
°F or °C	°F or °C
Range	89°-101°F or 31.5°-38.5°C
Case	Rolls of 2500 or 5000

Product Codes:

L215F
L215C

URINE SPECIMEN COLLECTION

SUBJECT I.D. _____

DATE _____ COLLECTORS INITIALS _____

Lot: 1295
- Read at Center of Line -

BriteLine® Write-on Label

Description	Disposable with Adhesive Label on Identification Write-on Label
°F or °C	°F or °C
Range	89°-101°F or 31.5°-38.5°C
Case	Rolls of 2000

Product Codes:

L216F
L216C

Product Highlights

- **Continuous Display Thermometer for Urine Specimen Collection Cup**
- **Easy to Read Moving Line Strip with Adhesive Backing**
- **Instant Response**
- **Conforms to SAMHSA Guidelines**
- **Roll Stock or Labels for Automated Application**
- **Temperature Range: 89° - 101°F or 31.5° - 38.5°C**
- **Offset with a Range of ±1°F / .6°C**



To Read:

1. Apply to cup.
2. Line moves along the scale.
3. Read temperature at center of line after line stops.