

Feverscan® Mini

Workforce & Mass Screening Forehead Thermometer

Feverscan® Mini is a personal disposable forehead thermometer specifically designed to be used for workforce and mass temperature screening measures, while assisting institutions with maintaining social distancing.

The Mini is based on proprietary color change technology that is used for monitoring patients during surgical procedures and was successfully used worldwide during the SARS epidemic to check for fevers in schools, offices, homes and at airports.


The Mini is composed of a series of discrete temperature events; each event shows a green color as the indicated temperature is attained.



- **Fast Continuous Display**
- **Adhesive Backed**
- **Self-Applied**
- **Easy to Read**
- **No Staff - Patient Contact**
- **Reduces Cross-Contamination Risk**
- **Disposable**
- **Latex Free**
- **Made in the USA**

Workforce and Mass Screening Thermometer

Item No.	P436-100
Description	Disposable thermometer with adhesive backing. 100 per dispenser
°F or °C	°F
Range	94°F-104°F

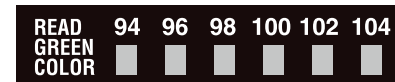

To Read:

1. Remove backing liner and apply monitor to the center of forehead, just above the eyebrows. Skin surface should be clean and dry.
2. Read monitor when the boxes stop changing color (generally 15 seconds) and while it is still on the forehead.
3. Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.

Please see IFU047 for complete instructions for use