

# Feverscan Reusable Product Data Sheet

**Product Description:**

Reusable Forehead Temperature Monitor

Product Codes: P432, P433

<b>Code #</b>	<b>Temperature Range</b>	<b>Core Adjusted Offset</b>
<b>P432</b>	94-104°F	4.0°F
<b>P433</b>	35-40°C	2.2°C

**Construction:**

Substrate	Polyester
Color Change Material	Non-toxic mixture of liquid crystal esters
Latex	Not made with natural rubber latex
Accuracy	± 1.0°F / 0.6°C
Storage & Handling	Store at 60 to 104°F in 15 to 95% RH Optimal storage at 60 to 80°F in 40 to 60% RH

**How to Read**

1. Apply to center of forehead just above the eyebrows and hold in place
2. Read monitor when the boxes stop changing color (generally 15 seconds), and while it is still on the forehead.
3. Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.

**Intended Use**

- Measure human forehead surface temperature and monitor temperature trends